Brio

Instructions for Use

BRIO30

EN	Instructions for Use	1
FR	Instructions D'Utilisation	7
DE	Gebrauchsanweisung	13
IT	Istruzioni per L'Uso	19
ES	Instrucciones de Uso	25
NO	Bruksanvisning	31
RU	Инструкция протезиста	37
TR	Kullanım Talimatları	43

Blatchford:

1 Description and Purpose

EN

Application:

These instructions are for use by the practitioner.

Brio is to be used exclusively as part of a lower limb prosthesis.

Brio is a push button operated manually adjustable ankle allowing the user to accommodate a variety of heel heights for shoe changes.

Note, the anterior aspect of the unit is marked to ensure correct orientation when fitting.

This device is recommended for users at Activity Level 1-4 and who may benefit from the ability to optimise the PF/DF alignment of their prosthetic foot within a range of heel heights.

Contra-indications:

This device may not be suitable for users who cannot reach their ankle or are not dextrous enough to operate the push button or those with poor cognitive function.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding maintenance.


When adjusting, please be aware of the potential finger trap hazard between the upper and lower parts of the device.

2 Construction

Principal parts:

- Top Housing (Aluminium)
- Lower Housing (Aluminium)
- Main Assembly (Steel, Stainless Steel, Aluminium, Nylon, Polyacetal Homopolymer, Hydraulic Oil)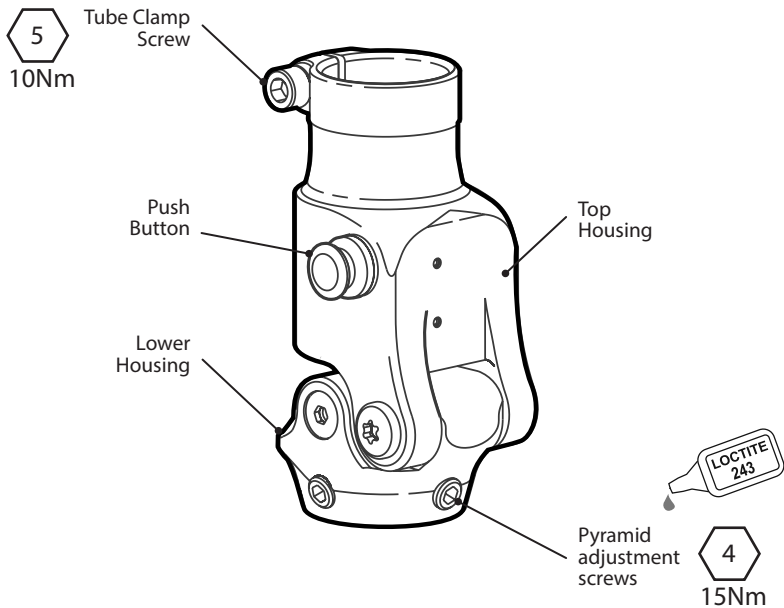

3 Function

Brio comprises a hydraulic body assembly containing a manually operated release valve. Operation of the release valve removes resistance to movement and allows adjustment of plantar flexion and dorsiflexion. The hydraulic body assembly is connected to a female pyramid lower housing via two pivot pins.

4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check that all attachment screws are secure, clean and reassemble as necessary

The user should be advised:

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Any unusual noise
- Reduced ankle support (free movement with release button not depressed)
- M-L movement or 'play' in pivots

The practitioner must also be informed of any changes in body weight and/or activity level.

Cleaning

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers. Thoroughly dry before use.

5 Limitations on Use:

Intended Life:

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:

Avoid exposing the product to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Suitable for outdoor use

Exclusively for use between -15°C to 50°C.

It is recommended that only Blatchford products be used in conjunction with the product.


6 Alignment and Set-up

When adjusting the Brio, please be aware of the potential finger trap hazard between the upper and lower parts of the device.

Depress the button to adjust, release to lock in position.



Set the device to maximum dorsiflexion (heel down, toe up). Align shin normal to the ground with lowest heel height (possibly barefoot) the user is likely to wear by adjusting the foot angle on the pyramid. This will provide the user with the maximum range to increase their heel height when required.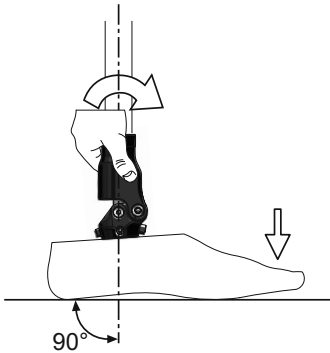

Instruct the user to confirm any alignment adjustment changes in a safe environment before walking.

Adjustment: *Load heel or toe as appropriate*



Instruct the user on the importance of maintaining the correct alignment and hand them the User Guide.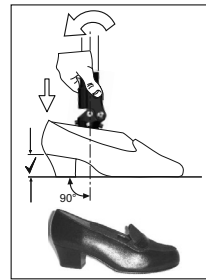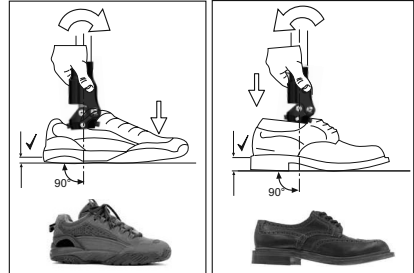

Finishing

Finish using a foam or Plastazote fairing and cut a suitable hole for access to the adjuster button. If required, cut a circular disc of an appropriate color material to adhere to the button.


Ensure button operation is not affected by the cosmetic covering.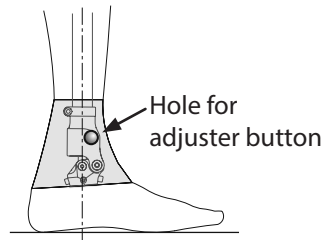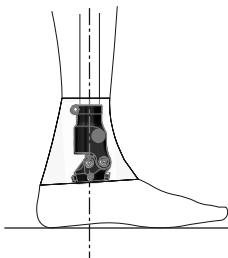

9 Technical Data

Operating and Storage Temperature Range:	-15°C to 50°C
Component Weight:	395g
Activity Level:	1-4
Maximum User Weight	
Activity Level 4	100kg
Activity Level 1-3	125kg
Proximal Alignment Attachment:	Ø 30mm Female Tube Clamp (Blatchford) Full 360° Rotation
Distal Alignment Attachment:	Female Pyramid (Blatchford)
Range of heel height adjustment: (Excludes additional range of motion provided by the heel and toe springs)	30mm
Build Height: [See diagram below]	107mm

Fitting Length

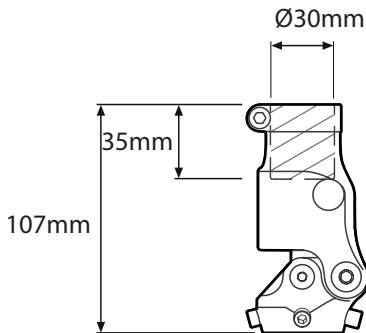Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

Brio is warranted for 24 months.

See Blatchford website for the current full warranty statement.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd