Brio

User Guide

BRIO30

EN	User Guide	1
FR	Guide de l'Utilisateur	3
DE	Bedienungsanleitung für Anwender	5
IT	Guida per il Paziente	7
ES	Guia de Usuario	9
NO	Brukerhåndbok	11
RU	Руководство пользователя	13
TR	Kullanım Kılavuzu	15

Brio





Please keep these instructions.

Brio is a push button operated manually adjustable ankle allowing the user to adjust heel height for shoe changes or bare foot walking.


When adjusting please be aware of the potential finger trap hazard between upper and lower parts of the device

Load heel or toe as appropriate. It is essential to maintain correct alignment. After adjustment always confirm stability before walking.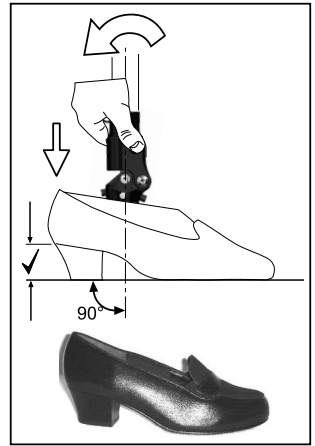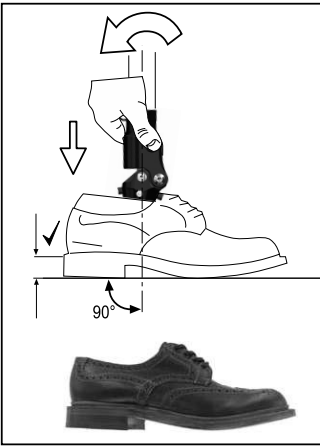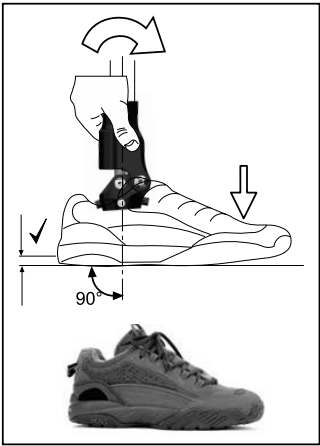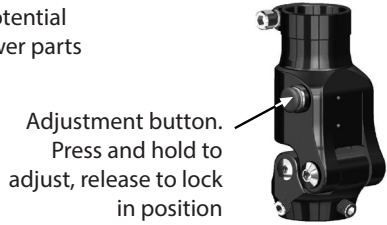

Environment:

Avoid exposing the Brio to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example, as these may promote premature wear. Exclusively for use between -15°C to 50°C.

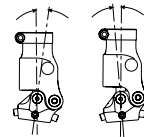Approx. range of heel height adjustment = 30mm

Suitable for outdoor use

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check that all attachment screws are secure, clean and reassemble as necessary

The user should be advised:

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Any unusual noise
- Reduced ankle support (free movement with release button not depressed)
- M-L movement or 'play' in pivots

The practitioner must also be informed of any changes in body weight and/or activity level.

Cleaning

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers. Thoroughly dry before use.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

Brio is warranted for 24 months.

See Blatchford website for the current full warranty statement.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Brio and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd