TT Pro

Instructions for Use

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EN

Application

These instructions are for the practitioner.

This device is for use as a component of a lower limb prosthesis.

TT Pro is a shock and torsion device that balances shock absorption and energy management. Reduced stress at the interface between the body and the prosthesis provides greater comfort for users.

Contra-indications

This device may not be suitable for use in competitive sports events, as this type of user may be better served by a specially designed prosthesis optimized for their needs.


Intended for single user.

Ensure that the user has understood all the instructions for use, drawing particular attention to the maintenance and safety information.

1.1 Torsion Rod and Spring Selection

Torsion Rod

- 1 = 1 dot
- 2 = 2 dots
- 3 = 3 dots

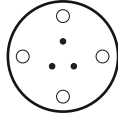Spring

- P = Purple
- W = White
- B = Black

Weight kg

Impact Level


	Low	Moderate	High
44-52	1P	2P	3W
53-59	2P	2W	3W
60-68	2P	3W	3B
69-77	2W	3W	3B
78-88	2W	3B	3B
89-100	3B	3B	3B
101-116	3B	3B	—
117-125	3B	3B	—

2 Construction


Principal Parts:

- Top Housing/ Tube Clamp Assembly (aluminum, carbon steel, stainless steel, acetal homopolymer, PTFE, PU)
- Torsion Rod (acetal homopolymer)
- Shaft/Locking Ring Assembly (stainless steel/aluminum alloy)
- Spring (carbon steel)
- Shin Assembly (stainless steel/aluminum)

TT Pro Pyramid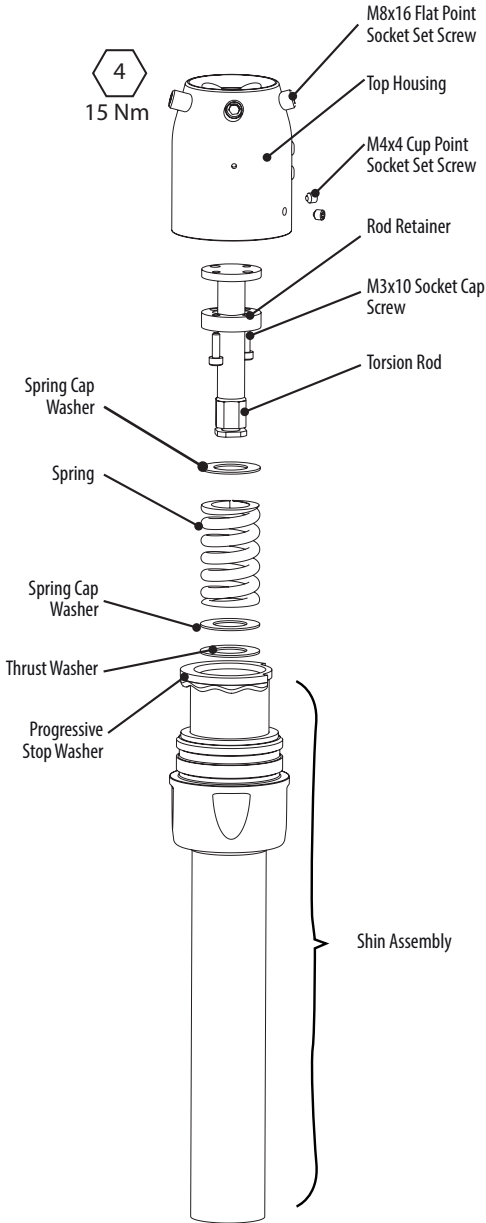

TT Pro Clamp

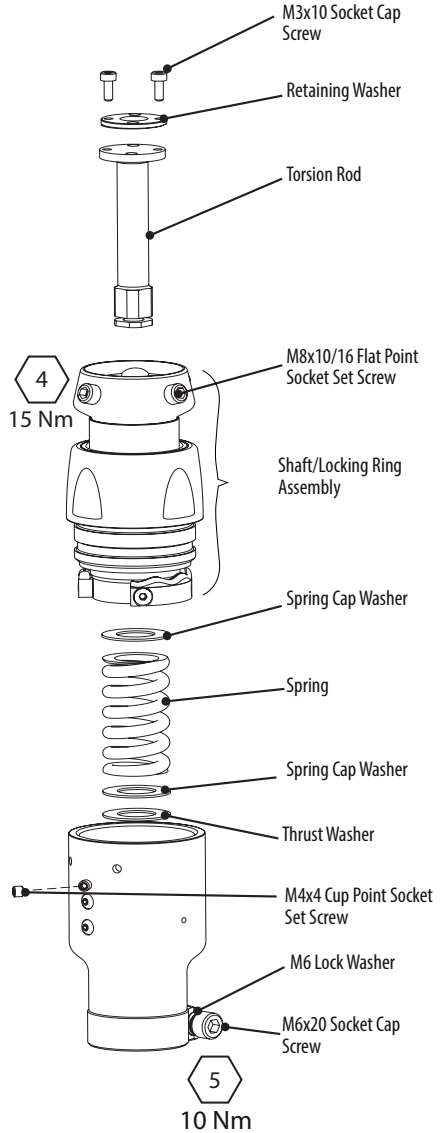3 Function

TT Pro comprises an energy efficient coil spring and a polymer torsion rod. Together, these components reduce both shear force at the socket interface during heel strike and torsional loads during turning. Energy that is stored in the axial spring during stance phase is returned at toe-off to give a smooth energy efficient gait. The bearing surfaces are coated with titanium nitride, which provides a hard-wearing low friction surface for moving parts.

4 Maintenance

Only competent personnel should carry out maintenance.

Carry out the following maintenance annually:

- Disassemble, clean, and lubricate parts as shown in sections 1–10
- Check for rotational play
- Replace the torsion rod if it's worn

Give the user the supplied user information card and advise them to report the following to the practitioner:

- Changes in body weight or activity level
- Changes in the performance of this device, for example:
 - Unusual noises or play

Cleaning

Use a damp cloth and mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

5 Limitations on Use

Intended Life

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

Environment

Avoid exposing this device to corrosive elements such as water, acids and other liquids. Avoid abrasive environments such as those containing sand, as these may promote premature wear.

Suitable for outdoor use


Exclusively for use between -15 °C and 50 °C (5°F and 122 °F).

We recommend using Blatchford products with this device.

6 Bench Alignment

6.1 Additional Components




Follow the alignment instructions that are supplied with any additional components.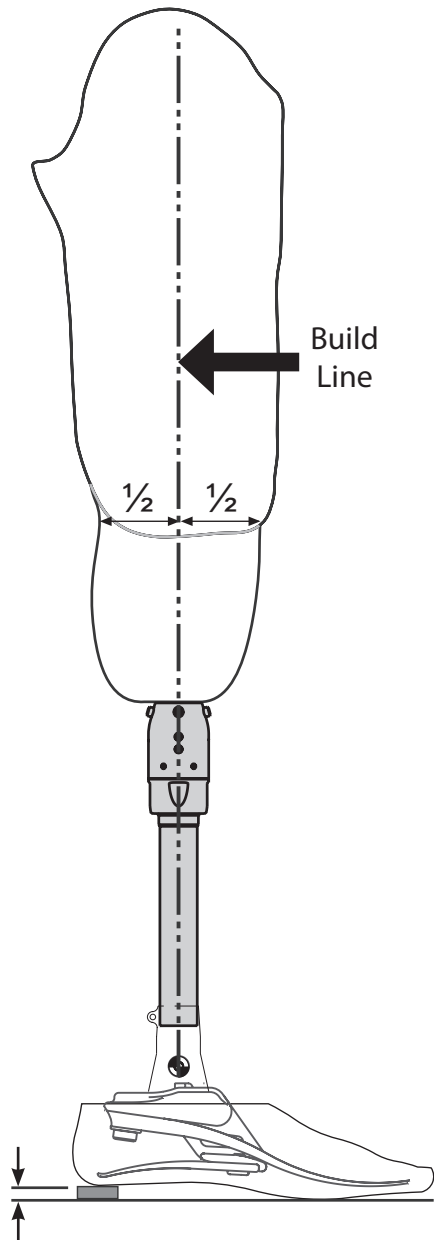







The build line should pass through the center of the device.

Allow for the user's footwear.




7 Dis-assembling TT Pro Pyramid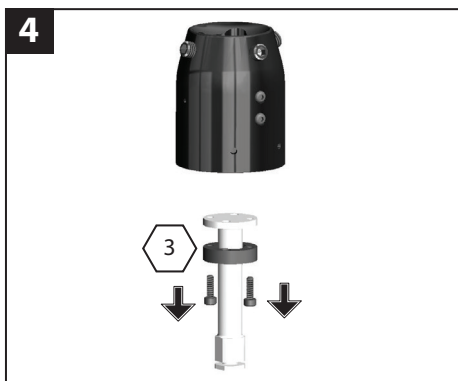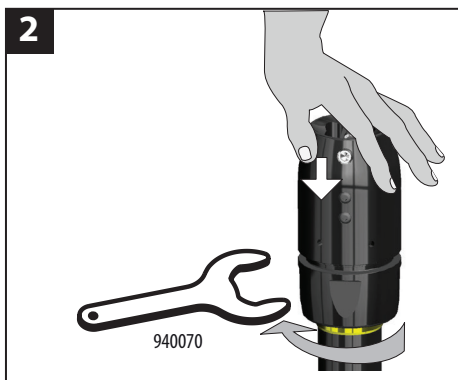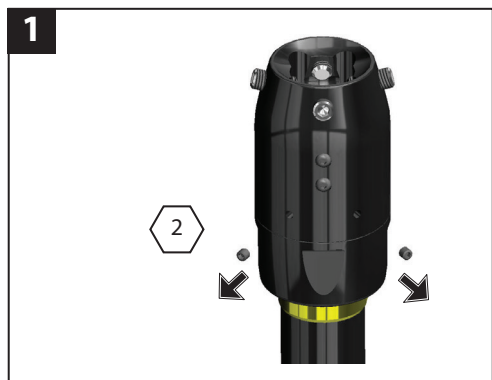







8 Assembling TT Pro Pyramid

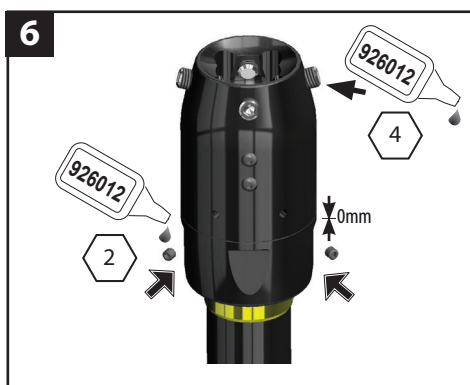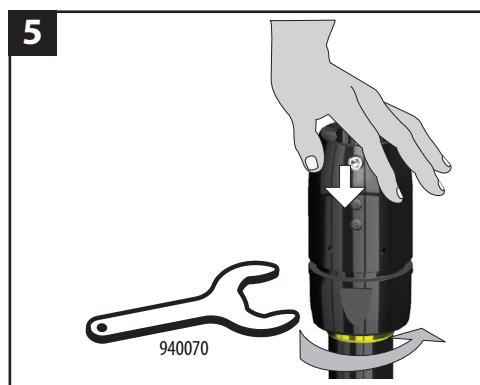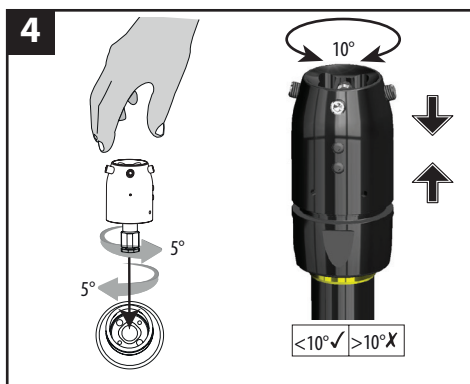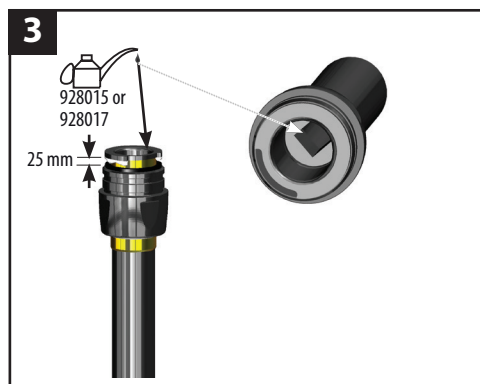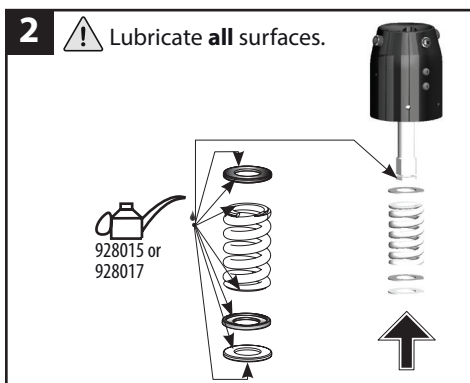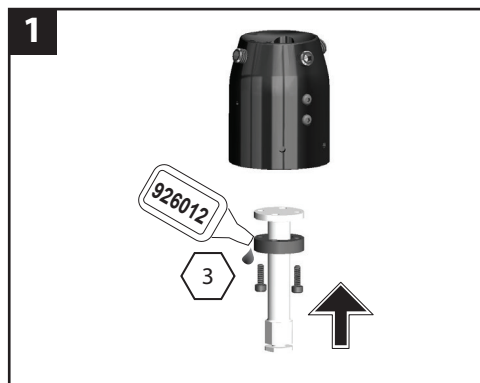9 Dis-assembling TT Pro Clamp

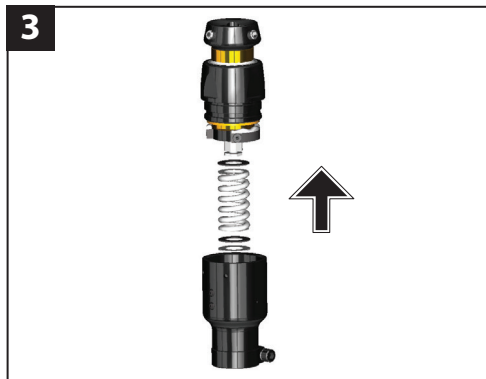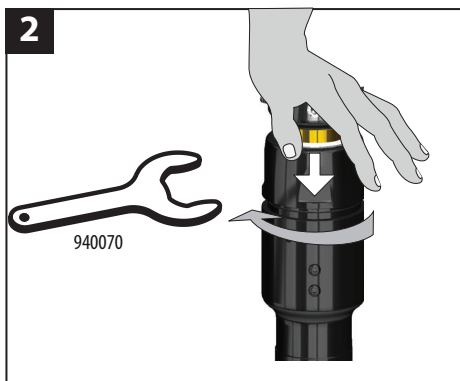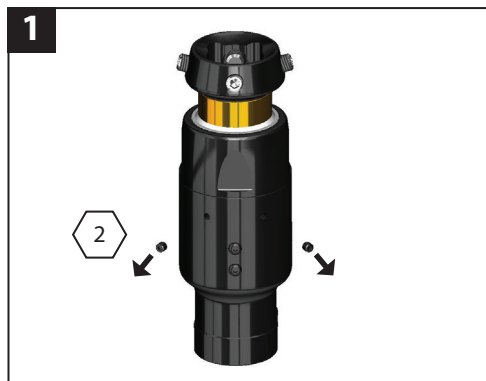10 Assembling TT Pro Clamp

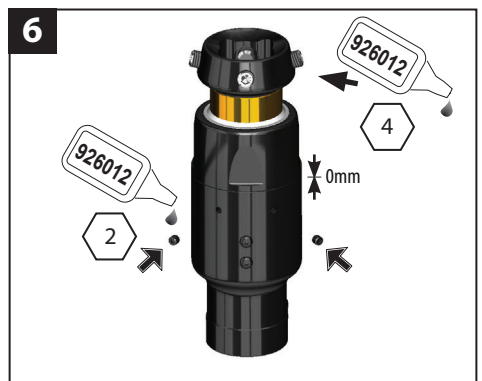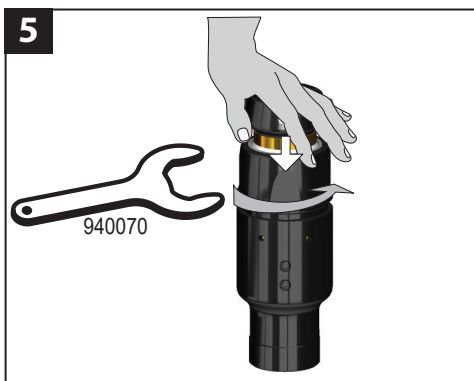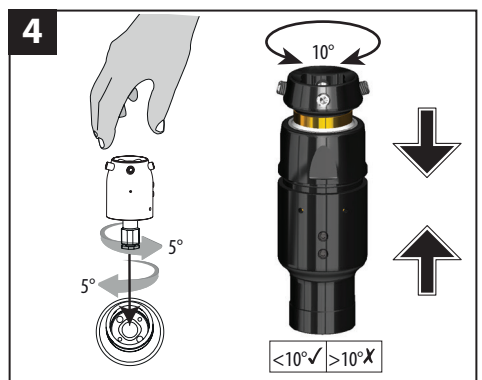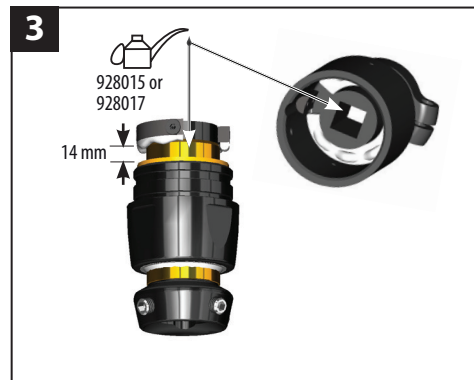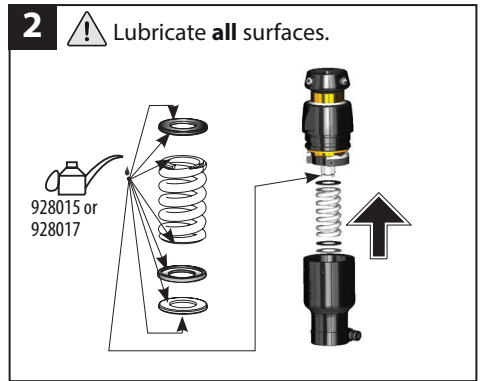
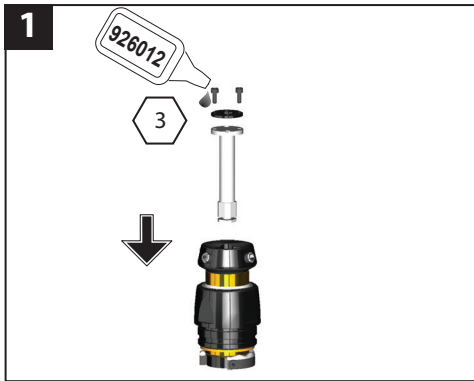
11 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight:	Pyramid — 523 g (18 oz) Clamp — 440 g (15 oz)
Activity Level:	1–4
Maximum User Weight:	Activity Level 1–3 — 125 kg (275 lb) Activity Level 4 — 100 kg (220 lb)
Range of Adjustment:	Proximal Attachment: $\pm 7^\circ$ Angular Distal Attachment: 360 ° Rotation
Build Height:	TT Pro Pyramid — 135–226 mm TT Pro Clamp — 109 mm
Proximal Attachment:	Female Pyramid (Blatchford)
Distal Attachment:	TT Pro Pyramid — Blatchford $\varnothing 30$ mm Tube Clamp TT Pro Clamp — Blatchford $\varnothing 30$ mm Tube


Fitting Length: TT Pro Pyramid

Note: Fit the device with the screws facing anteriorly.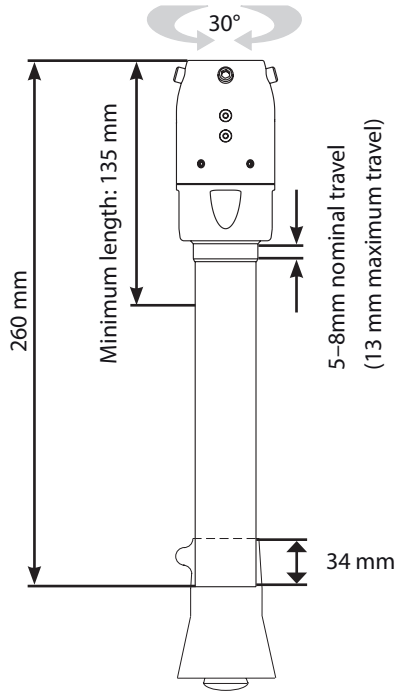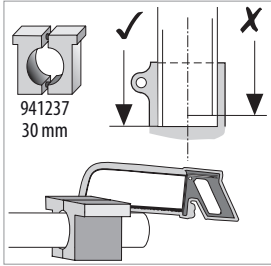

Fitting Length: TT Pro Clamp

Note: Fit the device with the screws facing anteriorly.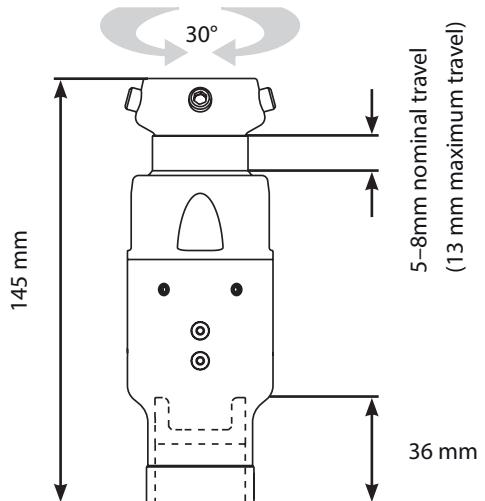


12 Ordering Information

Order example


TT PRO Pyramid

TTPRO-PYR3B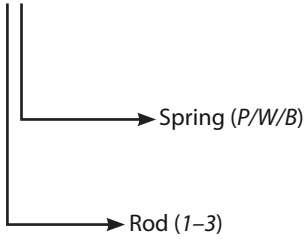

TT PRO Clamp

TTPRO-30-3B

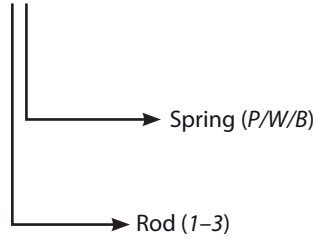Part	Part Number
Torsion Rod	406007 0.5Nm (1 dot)
	406008 0.6Nm (2 dots)
	406009 0.7Nm (3 dots)
Spring	406014 Purple (P)
	406015 White (W)
	406016 Black (B)

Part	Part Number
M8x16 Flat Point Socket Set Screw	910474
M8x10 Flat Point Socket Set Screw	910476
M3x10 Socket Cap Screw	910012
M4x4 Cup Point Socket Set Screw	910013
M6x20 Socket Cap Screw	910048
M6 Lock Washer	916150
Spring Cap Washer	406013
Thrust Washer	405031
Progressive Stop Washer	406004
30 mm Shin Tube Cutting Jig	941237
TT Locking Ring Spanner	940070

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

This device is warranted for 24 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Blatchford website for the current full warranty statement.

Environmental Aspects

This product is constructed from recyclable materials. Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd