# AqualimbTT

## Instructions for Use

629705L-629716L

EN	Instructions for Use	3
FR	Instructions D'Utilisation	11
DE	Gebrauchsanweisung	19
IT	Istruzioni per L'Uso	27
ES	Instrucciones de Uso	35
NO	Bruksanvisning	43
RU	Инструкция протезиста	51
TR	Kullanım Talimatları	59

**Blatchford:**

# 1 Description and Purpose

## Application

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to AqualimbTT.

Make sure that the user understands all the information in these instructions for use, particularly the maintenance and safety information.

This device is a shin-foot unit that is for use as a component of a trans-tibial prosthesis.

This device comprises a foot shell with an integrated shin for use in the shower.

## Activity Level

This device is for Activity Level 1 and 2 users. Activity Level 3 and 4 users may only use this device if they limit their activities to Activity Level 2.

## Contra-indications

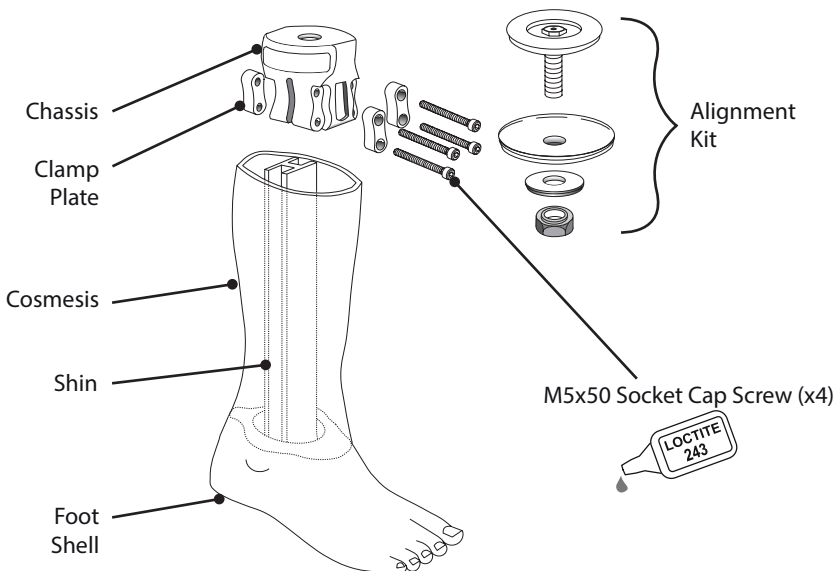
This device is for a single user.

This device is for barefoot use in the shower.

# 2 Construction

Principal parts:

- Cosmesis (PU foam)
- Foot Shell (PU foam)
- Shin (glass-fiber-reinforced nylon)
- Alignment Kit (stainless steel/aluminum/titanium/glass-fiber-reinforced nylon)
- Chassis (glass-fiber-reinforced nylon)
- M5x50 Socket Cap Screw (stainless steel)
- Clamp Plates (stainless steel)

---

## 3 Function

This device is a shin-foot unit that comprises a foot shell with an integrated shin. The shin can be shortened to suit the user's height. The sole of the foot features an anti-slip tread for grip on wet surfaces.

---

## 4 Maintenance

Only competent personnel should carry out maintenance.

Carry out the following maintenance annually:

- Make sure that the socket cap screws are tight. If they are loose, tighten them to the correct torque setting;
- Make sure that the anti-slip tread functions as intended, and replace if necessary.

Advise the user to report the following changes to the practitioner:

- Changes in body weight;
- Changes in the performance of the device, for example, unusual noises or play.

### Cleaning

Use a damp cloth and a mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

---

## 5 Limitations on Use

### Intended life

A local risk assessment should be carried out based upon activity and usage.

### Lifting loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof up to a maximum depth of 1 meter.

To prevent damage or wear, thoroughly rinse this device with fresh water after use in the following environments:

- Abrasive environments, such as those that contain sand or grit;
- Salt or chlorinated water.

Finish this device to prevent water ingress (see [section 7.1](#)). If water enters it, invert it to allow the water to drain, then leave it to dry.

This device is buoyant. If the user intends to submerge this device, remove the cosmesis with a suitable cutting tool (see [section 7.1](#), step 8a).

Suitable for submersion

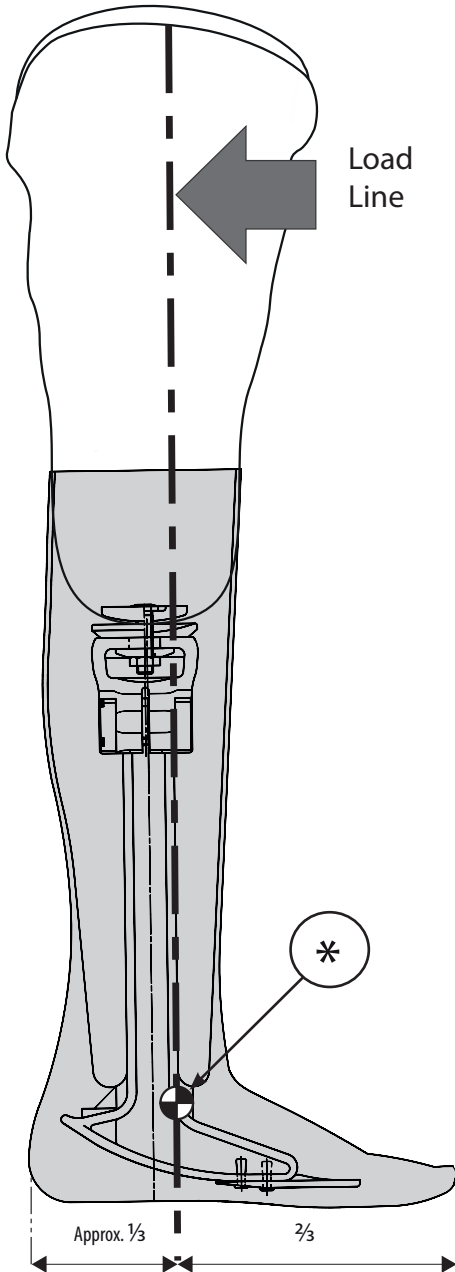
Take care on wet and slippery surfaces.

Exclusively for use between -15 °C and 50 °C (5 °F and 122 °F).

We recommend using Blatchford products with this device.

## 6 Alignment

### Static alignment

#### Sagittal plane

\* With the flexion accommodated, the load line should pass through the target shown in the image,  $\pm 10$  mm A-P. The target intersects the point where the anterior face of the shin meets the keel.

Make sure that when the user is standing, the heel and toe are evenly loaded and the foot is in full contact with the floor.

### Dynamic alignment

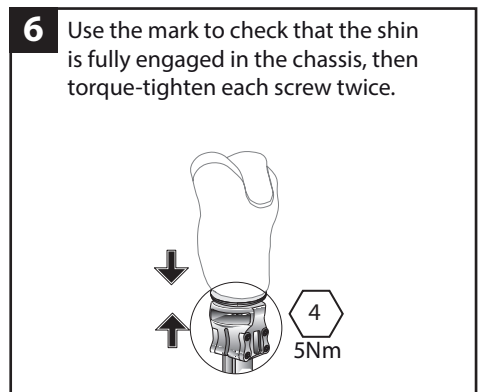
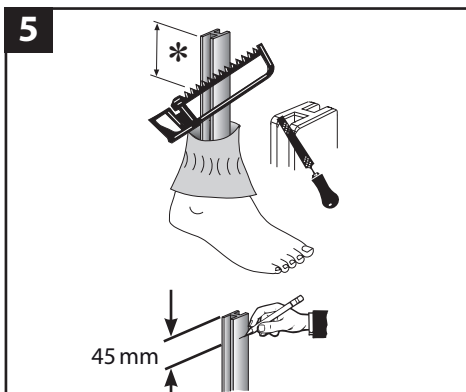
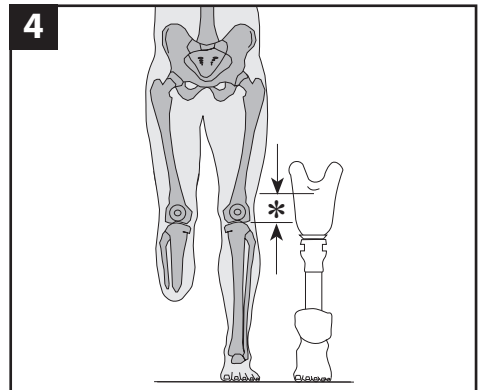
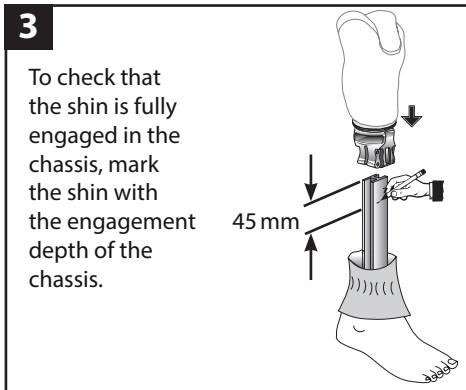
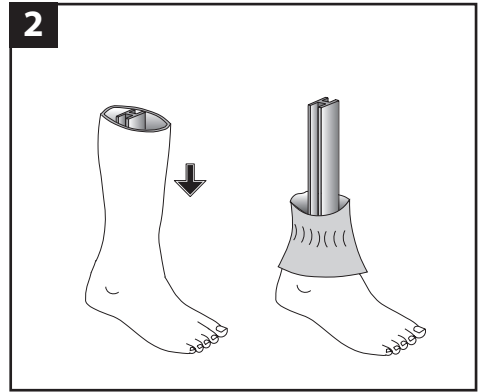
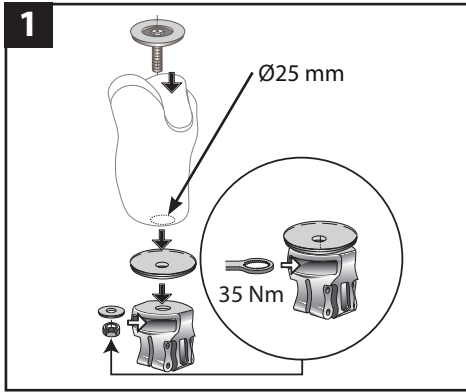
#### Coronal plane

Minimise M-L thrust by adjusting the relative positions of the socket and the foot .

#### Sagittal plane

Make sure that the user transitions smoothly from heel strike to toe-off.

# 7 Assembly

## 7.1 Assembly

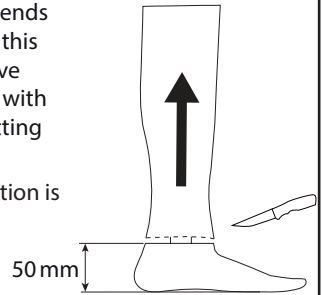
**7**

Go to *Alignment* (see section 6), then continue to step 8.

**8a**




If the user intends to submerge this device, remove the cosmesis with a suitable cutting tool.

**Note:** This action is irreversible.

**8b**

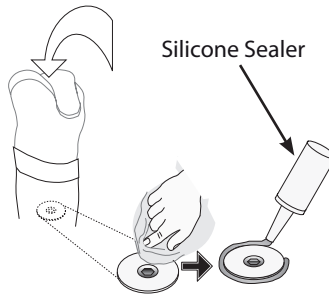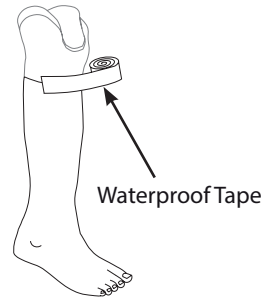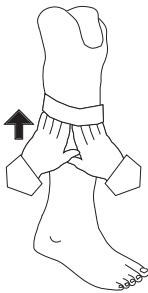If the user does not intend to submerge this device, finish it to prevent water ingress.


## 8 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (size 26):	1.39 kg (3 lb)
Activity Level:	1-2
Maximum User Weight:	100 kg (220 lb)
Build Height:	See the diagram below

### Fitting length

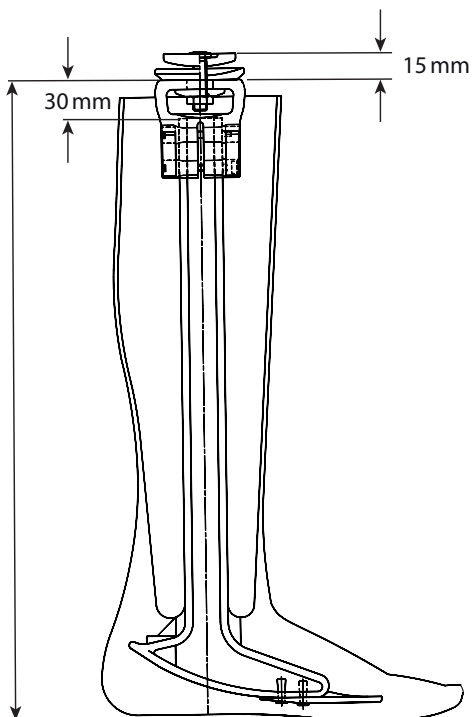450 mm maximum  
315 mm minimum  
130 mm minimum if the shin is removed  
(see [section 7.1](#), step 8a)

---

## 9 Ordering Information

### AqualimbTT

Size (cm)	Part Number — Left	Part Number — Right
22	629705L	629706L
23	629707L	629708L
24	629709L	629710L
25	629711L	629712L
26	629713L	629714L
27	629715L	629716L

### Replacement Parts

Part	Part Number
Alignment Kit	169106
M5x50 Socket Cap Screw	910415

**Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

**CE conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

**Warranty**

This device is warranted for 24 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

**Manufacturer's Registered Address**

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Blatchford is a registered trademark of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd