Echelon

User Guide

EC22L1S-EC30R8S
EC22L1SD-EC30R8SD

EN	User Guide	2
FR	Guide de l'Utilisateur	4
DE	Bedienungsanleitung für Anwender	6
IT	Guida per il Paziente	8
ES	Guia de Usuario	10

User Guide

EC22L1S–EC30R8S

Please keep these instructions.

The term *device* is used throughout this document to refer to Echelon.

This device is to be used exclusively as part of a lower limb prosthesis.

The device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve on postural sway and symmetry while easing abnormal pressures at the socket interface.

A moderate-energy-return foot with multi-axial ankle movement. Independent heel and toe springs provides some axial deflection. The split toe provides good ground compliance.

-  **After continuous use the ankle casing may become hot to the touch.**
-  **To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the foot shell must be used at all times.**
-  **Be aware of finger trap hazard at all times.**

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Softening of springs
- Excessive play in bearings
- Any unusual noise

The user should contact their practitioner if their condition changes.

The practitioner must also be informed of any changes in body weight and/or activity level.

A regular visual check of the foot is recommended. Signs of wear that may affect function should be reported to the practitioner (e.g. significant wear or excessive discoloration from long term exposure to UV).

Cleaning

Use a damp cloth and mild soap to clean outside surfaces. Do not use aggressive cleansers.

Maintenance

Maintenance must be carried out by competent personnel. It is recommended that maintenance is carried out annually.

Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Suitable for submersion

Exclusively for use between -15°C and 50°C (5°F to 122°F).

It is recommended that only Blatchford products be used in conjunction with the device.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The device is warranted for 36 months - foot shell 12 months - glide sock 3 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Echelon and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd.