# Elite Blade

## Instructions for Use

EB24L1-EB30R9  
EB24L1D-EB30R9D

EN	Instructions for Use	2
FR	Instructions D'Utilisation	14
DE	Gebrauchsanweisung	27
IT	Istruzioni per L'Uso	41
ES	Instrucciones de Uso	55

**Blatchford:**

# 1 Description and Purpose

## Application

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to Elite Blade.

This device is to be used exclusively as part of a lower limb prosthesis.

A lightweight, high-energy-return foot, ideal for high impact use from the casual jogger to the serious runner. The flexible extended pylon reduces shock load transmission making it suitable for work, leisure and sports activities. Independent heel and toe springs provide axial deflection. The split toe provides good ground compliance.

This device is recommended for users that have the potential to achieve Activity Level 3 or 4. Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances and any such decision should be made with sound and thorough justification.

Appropriate footwear must be worn to avoid the risk of slipping in wet environments

## Contra-indications

This device may not be suitable for Activity Level 1 individuals or for competitive sports events, as these types of users will be better served by a specially designed prosthesis optimized for their needs.

Intended for a single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding maintenance.

## Spring set selection

Impact	Activity	User weight										kg lb
		44-52 (100-115)	53-59 (116-130)	60-68 (131-150)	69-77 (151-170)	78-88 (171-195)	89-100 (196-220)	101-116 (221-255)	117-130 (256-285)	131-147 (286-325)	148-166 (326-365)	
Low	3	1	1	2	3	4	5	6	7	8	9	Foot spring set
Mod	4	1	2	3	4	5	6	7	8	9		
High	4	2	3	4	5	6	7	8	9			
Low	Daily walking and occasional sports such as golf and hiking											
Moderate	Aggressive walking, frequent or daily sports such as jogging											
High	Daily activities such as distance running, climbing, lifting and carrying heavy objects for vocational purposes											

**Important: For higher impact users, do not exceed the weight limit for individual springs.**

### Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Note:

If in doubt choosing between two categories, choose the higher rate spring set.

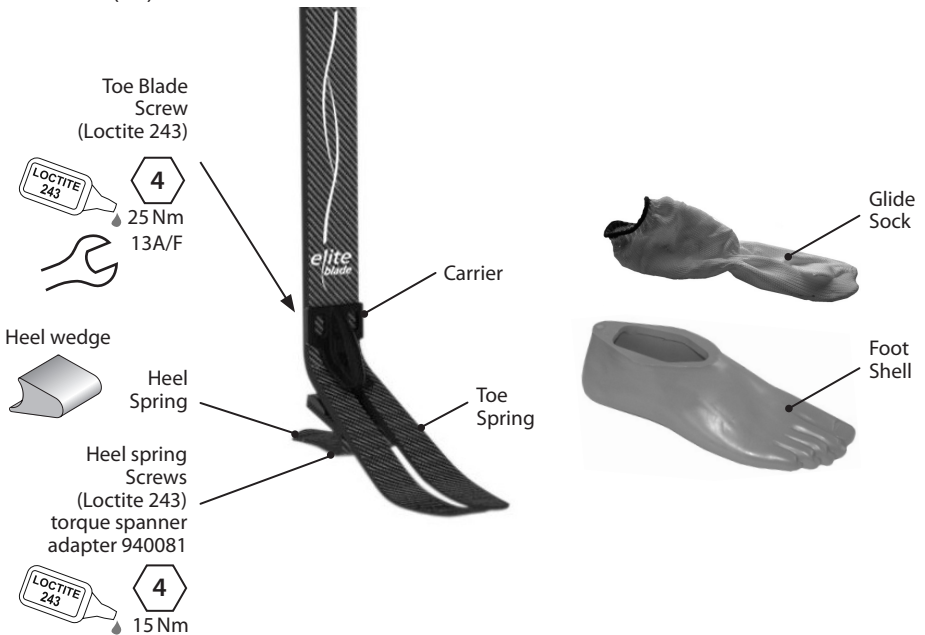
Foot spring set recommendations shown are for trans-tibial users.

For trans-femoral users we suggest selecting a spring set one category lower, refer to fitting advice Section 9 to ensure satisfactory function and range of movement

## 2 Construction

Principal parts:

- Carrier (aluminum)
- Heel & Toe Springs (e-carbon)
- Spring Attachment Screws (titanium)
- Glide Sock (UHM PE)
- Foot Shell (PU)

---

## 3 Function

This device comprises an e-carbon toe spring pylon and independent heel spring. The pylon can be cut to length to suit individual users.

Heel and toe springs are attached to the carrier assembly using titanium screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

---

## 4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Remove the foot shell and glide sock, check for damage or wear and replace if necessary.
- Check all screws for tightness, clean and reassemble as necessary.
- Visually check heel spring and toe blade for signs of delamination or wear and replace if necessary. Some minor surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The user must be handed the user information card supplied, and be advised of the following:

- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return or unusual noises.
- The practitioner must also be informed of any changes in body weight and/or activity level.

If this device is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

---

## 5 Limitations on use:

### Intended life

A local risk assessment should be carried out based upon activity and usage.

### Lifting loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

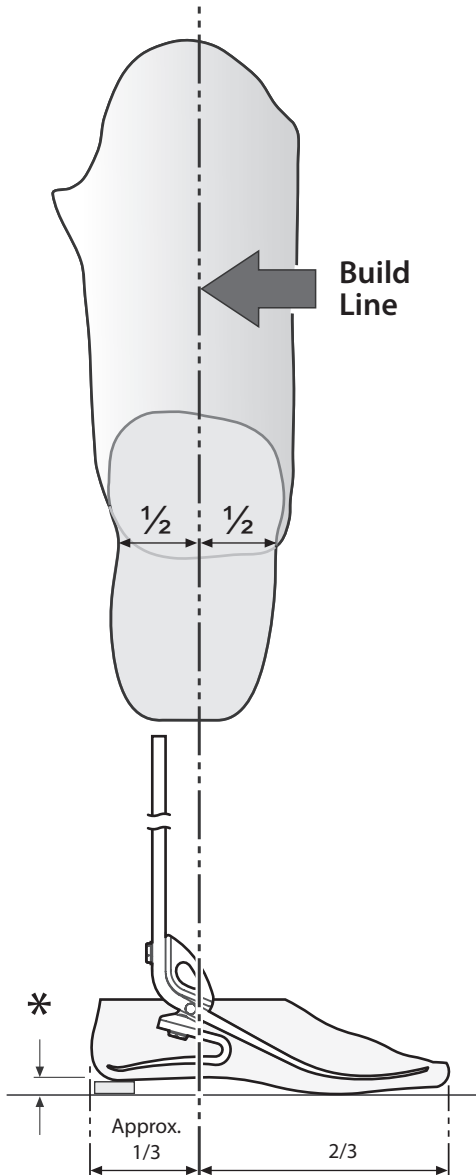
Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).

We recommend using Blatchford products with this device.

Suitable for submersion

## 6 Bench Alignment

### Static alignment

Setup length

Note: Shin blade should be vertical when used with a 10 mm heel height shoe. Lower heel height shoes may require the use of a packing piece below the heel.

Cut shin to approximate length, allowing 10–15 mm for final trimming. Note trial length will typically include 5 mm to allow for axial compression of heel and toe.

Build line

This should fall 1/3 of the foot length from the heel.

### Dynamic alignment

Coronal plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

Sagittal plane

Check for smooth transition from heel strike to toe-off. Ensure also that when standing the heel and toe are evenly loaded and that both are touching the floor.

### Trans-femoral alignment

Align trans-femoral components according to fitting instructions supplied with the knee, keeping the build line relative to this device as shown.

\* Allow for users own footwear

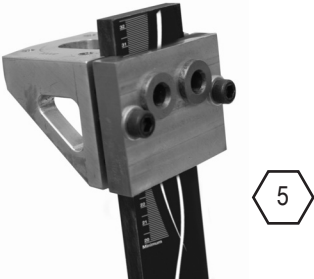
## 7 Cutting Shin to Length

**Note:** The following instructions assume the use of the male, sliding, rotating pyramid. If an alternative adapter is required, this should be taken into account throughout the build process.

Use appropriate health and safety equipment at all times including extraction facilities

**1**

Cut the blade 10–15 mm longer than required and assemble fitting clamp (941255).

**2**

Fit socket to fitting clamp top housing and tighten pyramid screws (15 Nm). Refer to alignment page for initial setup.

**3**

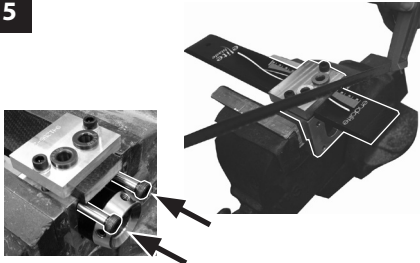
Tighten fitting clamp bolts to 15 Nm (check each screw at least twice).

**4**

Stand user, carry out dynamic alignment and walking trials. Adjust height to achieve definitive length with weight evenly distributed.

**5**

Remove socket, place fitting clamp in the vice and cut blade to length. Fit shoulder screws to prevent damage to components


**6**

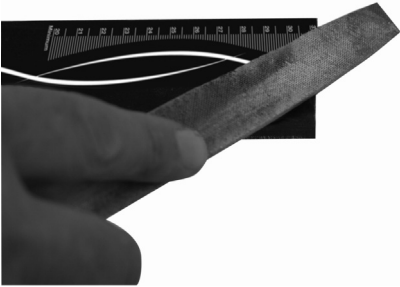Drill two 8.1 diameter holes through drill bushes.

## 7 Cutting Shin to Length *(continued)*

7

Deburr end of blade.

8

Slide pyramid into top housing (or fit preferred alignment option).

9

Loctite (243) & tighten (25 Nm) top housing bolts.  
Attach and align socket.

10

If a foam cosmesis is to be fitted, roughen the top surface of the foot shell to provide a bonding surface.  
Bond foam cosmesis to upper surface of the foot shell using Thixofix adhesive (926204) or equivalent and shape to suit.

# 8 Foot Shell Removal & Blade/Spring Replacement

**1**

4

Remove top housing from blade.

**2**

4

Remove lower carrier bolt.

**3**

Remove blade from carrier/foot shell.

**4**

Rotate heel/carrier assembly from foot shell/  
glide sock.

**5**

Remove heel spring from carrier.  
(13 mm spanner 940273)

**6**

Fit new heel spring, Loctite (243) & tighten bolts  
(15 Nm). Use torque spanner attachment 940081)

# 8 Foot Shell Removal & Blade/Spring Replacement (cont.)

**7**

Attach blade to heel carrier. Loctite (243) & tighten bolt (25 Nm).

**8**

Fit glide sock and lubricate foot shell with 928017.

**9**

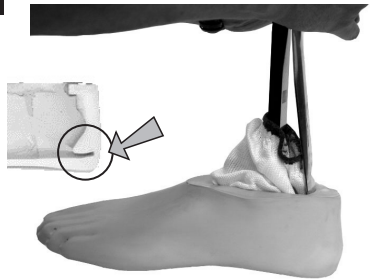
Fit glide sock and lubricate foot shell with 928017. Fit assembly into foot shell, engage toe of blade into foot shell.

**10**

Using a lever, ease assembly into shell.

**11**

Ensure heel spring is engaged into slot.

**12**

Cut blade using method shown previously before attaching top housing. Loctite (243) & tighten bolts (25 Nm). Re-apply cosmesis if applicable.

---

## 9 Fitting Advice

Springs are supplied as matched sets i.e. the blade/toe and heel springs are designed to work together to give smooth progression for most users.

### Heel wedge

A heel wedge is supplied with the foot. Fitting the wedge will have the effect of stiffening the heel spring. These can be taped in place for trial. For permanent fitting, wedges should be adhered in place by application of Loctite 424 (926104) between the lower contacting surface of the heel and the wedge.

	Symptoms	Remedy
<b>Heel too soft</b>	<ul style="list-style-type: none"><li>• Sinking at heel strike</li><li>• Difficulty climbing over the toe (toe feels too hard)</li></ul>	<ol style="list-style-type: none"><li>1 Move socket anteriorly in relation to the foot (excess movement may result in drop off)</li><li>2 Add heel wedge</li></ol>
<b>Heel too hard</b>	<ul style="list-style-type: none"><li>• Rapid transition from heel strike through stance phase</li><li>• Difficulty in controlling heel action, foot jars into mid-stance</li><li>• Foot feels too rigid</li></ul>	<ol style="list-style-type: none"><li>1 Remove heel wedge (if fitted)</li><li>2 Move socket posteriorly in relation to foot</li></ol>
<b>Toe/ Blade too soft</b>	<ul style="list-style-type: none"><li>• Rapid progression through mid-stance.</li><li>• 'Drop off' at higher Activity Levels</li></ul>	<ol style="list-style-type: none"><li>1 Move socket posteriorly in relation to foot</li><li>2 Plantarflex foot slightly - note: some re-alignment may be required</li></ol>

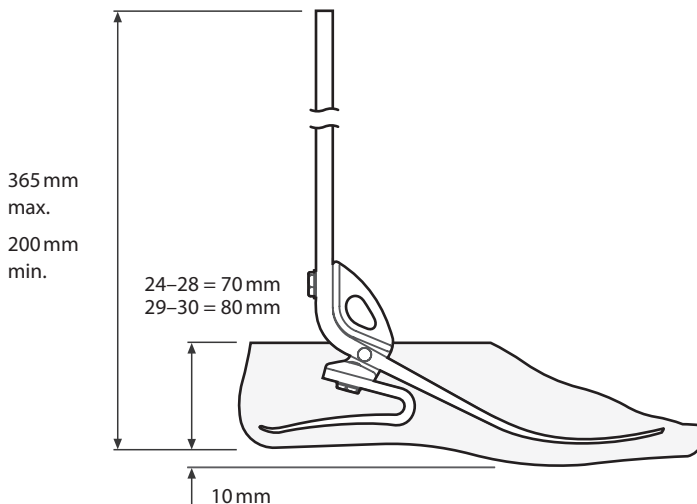
Please contact your supplier if it is not possible to achieve a smooth gait after following the advice above.

---

## 10 Technical Data


Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (size 26):	580 g (1 lb 5 oz)
Activity Level:	3–4
Maximum User Weight:	166 kg (365 lb)
Proximal Alignment Attachment:	Available with separately supplied alignment devices
Build Height: (See diagram below)	200–365 mm
Heel Height	10 mm

### Fitting length

# 11 Ordering Information
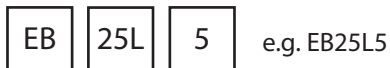
## Order example

Size Spring Set  
Side Category

Available from size 24 to size 30:

EB24L1 to EB30R9

EB24L1D to EB30R9D

(add 'D' for a dark tone foot shell)

## Spring Kits

Rate	Foot Sizes		
	24–26	27–28	29–30
Set 1	539610	539619	Special order
Set 2	539611	539620	Special order
Set 3	539612	539621	539630
Set 4	539613	539622	539631
Set 5	539614	539623	539632
Set 6	539615	539624	539633
Set 7	539616	539625	539634
Set 8	539617	539626	539635
Set 9	539618	539627	539636

## Foot Shell

For dark add suffix D

Left	
24L	539005
25L	539007
26L	539009
27L	539011
28L	539013
29L	539015
30L	539017

Right	
24R	539006
25R	539008
26R	539010
27R	539012
28R	539014
29R	539016
30R	539018

## Glide Sock

One size fits all Part No. 532811

Fitting Clamp Part No. 941255

**Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

**CE conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

**Warranty**

This device is warranted for 36 months - foot shell 12 months - glide sock 3 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

**Trademark Acknowledgements**

Elite Foot and Blatchford are registered trademarks of Blatchford Products Limited.

**Manufacturer's Registered Address**

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH.

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Blatchford SAS
Endolite India Ltd