# VTM

## Instructions for Use

VTM1 – VTM4

EN	Instructions for Use	2
FR	Instructions D'Utilisation	11
DE	Gebrauchsanweisung	20
IT	Istruzioni per L'Uso	29
ES	Instrucciones de Uso	38

# 1 Description and purpose

## Application:

These instructions are for use by the Clinician/Practitioner.

The VTM (Vector Thrust Male) is to be used exclusively as part of a lower limb prosthesis.

- The device provides axial and rotational shock absorption and energy return.
- The device is recommended for users that have the potential to achieve up to K3/K4 activity use and who may benefit from a reduction in shear loads at the residuum/socket interface.
- For new fittings the Elite blade needs to be cut to length. Instructions on how to do this are supplied with the cutting and drilling jig Part No. 941256.

## Contra-indications

There are no known contra-indications if used in accordance with these instructions.

Intended for single user.

Ensure that the user has understood all Instructions for use, drawing particular attention to the notes under Maintenance.

## Product Code

- VTM1 Fitted with Spring Rate 1
- VTM2 Fitted with Spring Rate 2
- VTM3 Fitted with Spring Rate 3
- VTM4 Fitted with Spring Rate 4

## Axial Spring Selection Chart

		User Weight									
Impact	lb	100-115	116-130	131-150	151-170	171-195	196-220	221-255	256-285	286-325	326-365
	kg	44-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147	148-166
LOW		1	1	1	2	2	3	3	4	4	4
MODERATE		1	1	2	2	3	3	4	4	4	
HIGH		1	2	2	3	3	4	4	4		
		Axial spring rating									

### Impact levels:

**Low** Daily walking and occasional sports such as golf and hiking.

**Moderate** Aggressive walking, frequent or daily sports such as jogging.

**High** Daily activities such as distance running, climbing, lifting and carrying heavy objects for vocational purposes.


**Extreme** Rigorous daily walking at high cadence, competitive athlete in track and field type events, has vocation that requires jumping or loading of the prosthesis beyond normal levels.  
(extreme users should contact our sales team)

## 2 Construction

### Principal Parts:

- Housing Aluminium Alloy
- Pyramid Tube Titanium
- Axial Spring Titanium
- Screws Titanium/Stainless Steel/Nylon
- Bearings Reinforced Polymer

### VTM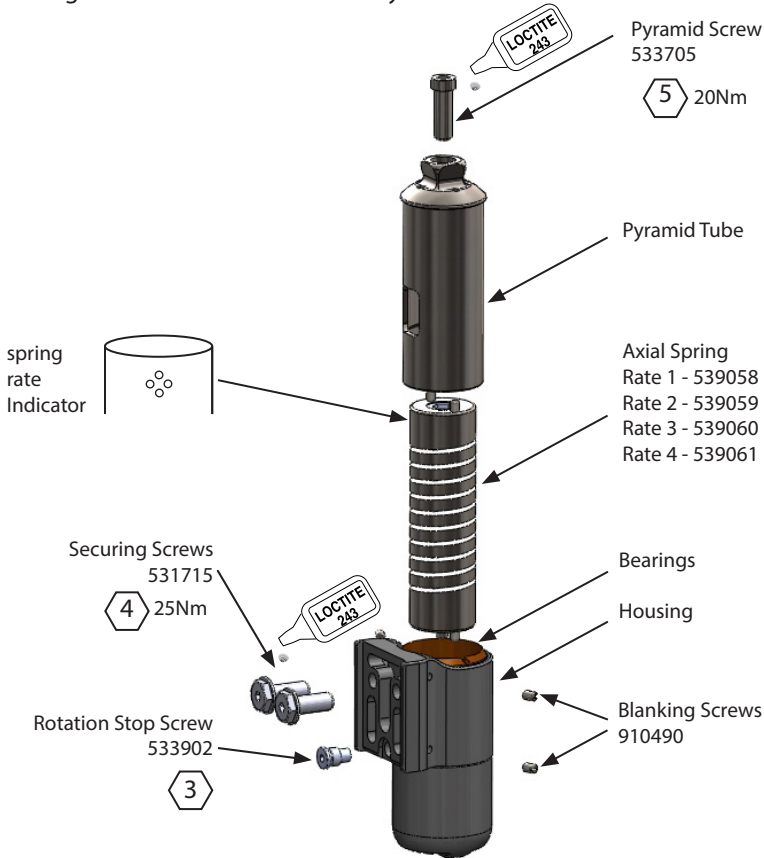

## 3 Function

The VTM comprises a housing assembly which contains a spring component that allows the Pyramid Tube to deflect and/or rotate through a limited range under an axial and/or torsional load. When the load is removed the spring returns the Pyramid Tube to its original position. The effect of this movement is to attenuate any shock forces that would otherwise be transmitted to the stump socket interface both in axial and rotational directions.

---

## 4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that annually the device is checked for visual defects that may affect proper function and that the bearing surface is re-lubricated. Use 'Sapphire Endure' (928015) or equivalent PTFE loaded grease and follow the instructions in Section 7 for dismantling and re-assembly.

The wearer should be advised:

The user should be advised to contact their Clinician/Practitioner if their condition changes.

Any changes in performance of the device must be reported to the Clinician/Practitioner.

Changes in performance may include:

- Increased or reduced axial or torsional stiffness
- Excessive play in bearings
- Any unusual noises

Cleaning:

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers.

---

## 5 Limitations on use

### **Intended Life:**

Service life of the product is covered by the warranty period.

A local risk assessment should be carried out based upon activity and usage.

This product is recommended for use with other Blatchford Products.

### **Lifting Loads:**

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### **Environment:**


Avoid exposing the VTM to corrosive elements such as water, acids and other liquids. Also avoid exposure to abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use and storage between temperatures of -10°C and 50°C [14°F and 122°F]

---

## 6 Alignment and Set-Up

### **Proximal alignment range**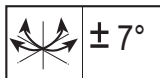

### **Fitting advice:**


VTM springs are designed to work together with the heel and toe springs of the Elite Blade to give smooth progression for most users.

The axial spring controls both compressive and torsional resilience. Typical axial movement should be between 3-6mm for normal walking. Alternative stiffness springs are available if required.

The VTM is designed to be interchangeable with existing Elite Blade Top Housings that have been fitted with the shifting Male Pyramid option (Part No. 539501 or 539601). No further length adjustment should be necessary.

For additional fitting advice see Elite Blade Instructions for Use (Part No. 938295).

## Alignment Diagram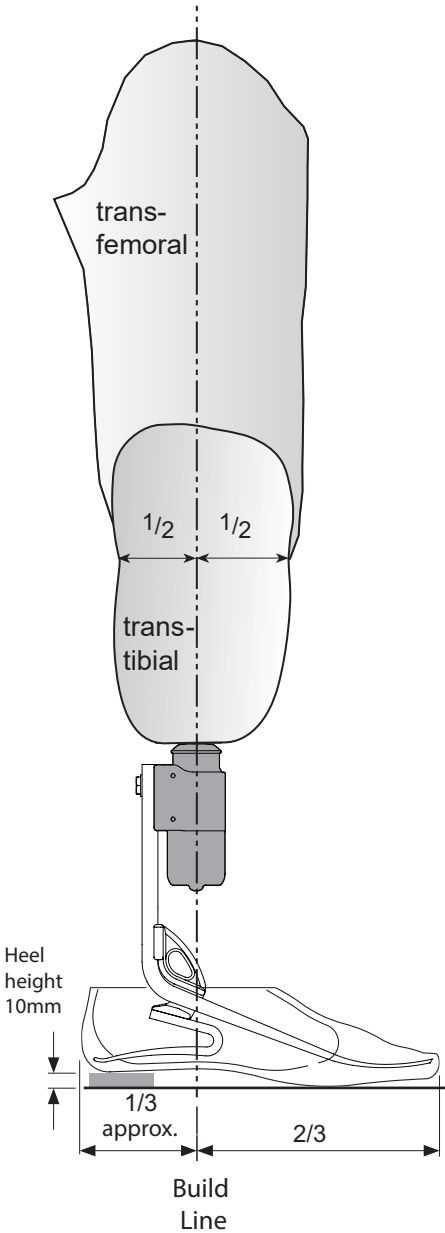

## Static alignment

Set Up Length

Note:

**Shin blade should be vertical when used with a 10mm heel height shoe.**

Lower heel height shoes may require the use of a packing piece below the heel. Cut shin to approximate length, allowing 10-15mm for final trimming. Note trial length will typically include 10mm to allow for axial compression of heel, toe and spring (axial).

Use a VT Cutting and Drilling Jig (Part No. 941256) to fit the VTM to an Elite Blade.

Build Line

This should fall 1/3 of the foot length from the heel (AP view shown).

## Dynamic alignment

Coronal Plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

Sagittal Plane





Check for smooth transition from heel strike to toe off. Ensure also that when standing the heel and toe are evenly loaded and that both are touching the floor.

Trans-femoral Alignment

Align trans-femoral devices according to fitting instructions supplied with the knee, keeping the build line relative to the VTM as shown.

# 7 Axial Spring Replacement

Remove VTM from the Elite Blade. Lubrication of the bearing surface may be carried out using 'Sapphire Endure' (928015) or equivalent PTFE loaded grease.

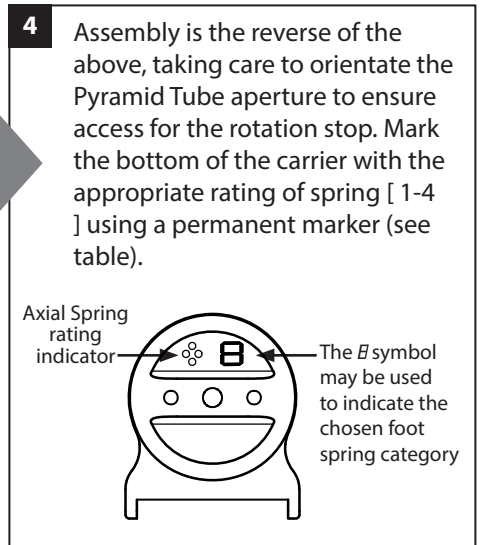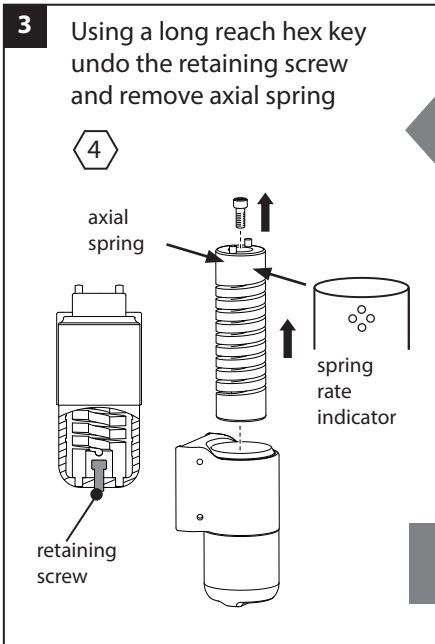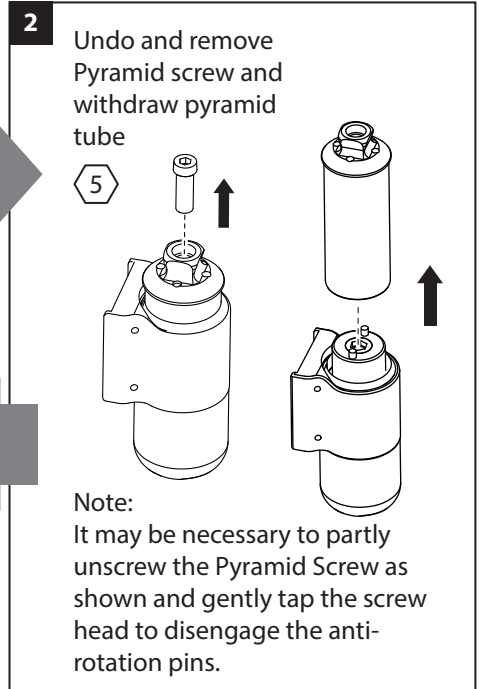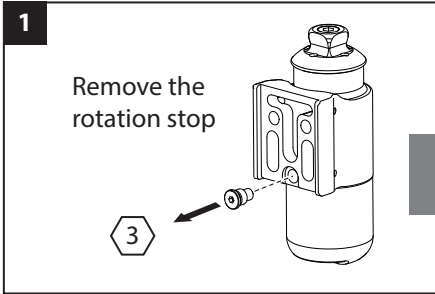Indicator	Spring Rate	Spring (Part No.)
	1	539058
	2	539059
	3	539060
	4	539061


---

## 8 Technical Specification

Part No. Range	VTM1 - VTM4
<b>Materials:</b>	Aluminium Alloy, Titanium, Reinforced Polymer
Operating and Storage Temperature Range:	-10°C to 50°C 14°F to 122°F
Component Weight:	450g ( 16ozs)
Recommended Activity Level:	3/4
Maximum User Weight:	166kg (365lb)
Proximal Alignment attachment:	Male Pyramid (Blatchford)
Maximum Rotation	± 15 degrees
Build Height Range [See diagram below]:	Minimum 215mm Maximum 380mm

---

### Fitting Length:

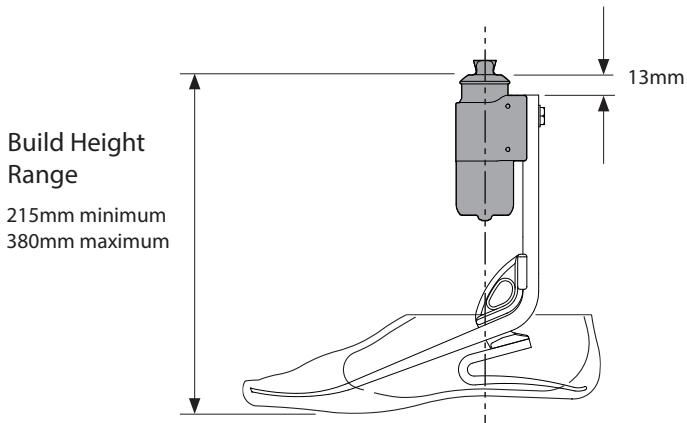## **Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

## **CE Conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

## **Warranty**

The VTM is warranted for 3 years. See Blatchford catalogue for details.

The user should be aware that changes or modifications not expressly approved will void the warranty.

## **Manufacturer's Registered Address**

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Blatchford SAS
Endolite India Ltd