SuperSACH

Instructions for Use

P529101-P529114

EN	Instructions for Use	3
FR	Instructions D'Utilisation	9
DE	Gebrauchsanweisung	15
IT	Istruzioni per L'Uso	21
ES	Instrucciones de Uso	27
NO	Bruksanvisning	33
RU	Инструкция протезиста	39
TR	Kullanım Talimatları	45

Blatchford:

1 Description and Purpose

SuperSACH provides the user with a cushioned heel and a flexible keel. The adjustable heel screw allows heel cushion adjustment for an individual fit.

Application

These instructions are for the practitioner.

SuperSACH is to be used exclusively as part of a lower limb prosthesis with an interfacing Blatchford product.

Activity Level

SuperSACH is recommended for Activity Level 1 and 2 users.

There are exceptions to our recommendations and we want to allow for individual circumstances — there may be Activity Level 3 and 4* users who may benefit from this

product, but this decision should be made with sound and thorough justification.

*Maximum user weight: 100 kg (220 lb).

Contra-indications


SuperSACH may not be suitable for use in competitive sports events, as these types of users may be better served by a specially designed prosthesis optimized for their needs.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to [section 4](#) regarding maintenance.

 Appropriate footwear must be worn to avoid the risk of slipping in wet environments.

2 Construction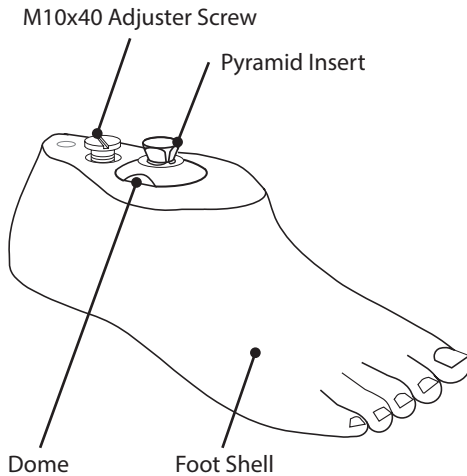

Principal parts:

- Foot Shell (PU foam)
- M10x40 Adjuster Screw (nylon)
- Pyramid Insert (stainless steel)
- Keel (nylon ST)
- Dome (aluminium alloy)

3 Function

The SuperSACH foot comprises a lightweight thermoplastic keel which gives a gentle rolling action through stance. The keel lies within a standard closed foot shell profile and features a sandal toe for a more natural appearance. An adjuster screw in the heel allows the user to customize the hardness of the heel cushion for an individual fit.

4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check for signs of wear and tear

Advise the user to report the following to the practitioner:

- Changes in body weight or activity level
- Changes in the performance of this device — changes may include:
 - Unusual noises or play

Cleaning

Use a damp cloth and mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

5 Limitations on Use

Intended Life

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

Environment

This device is waterproof up to a maximum depth of 1 meter. To prevent wear or damage to moving parts, thoroughly rinse with fresh water after use in:

- Abrasive environments such as those that contain sand or grit
- Salt or chlorinated water


Suitable for submersion

Exclusively for use between -15 °C and 50 °C (5 °F and 122 °F).

We recommend using Blatchford products with SuperSACH.

6 Bench Alignment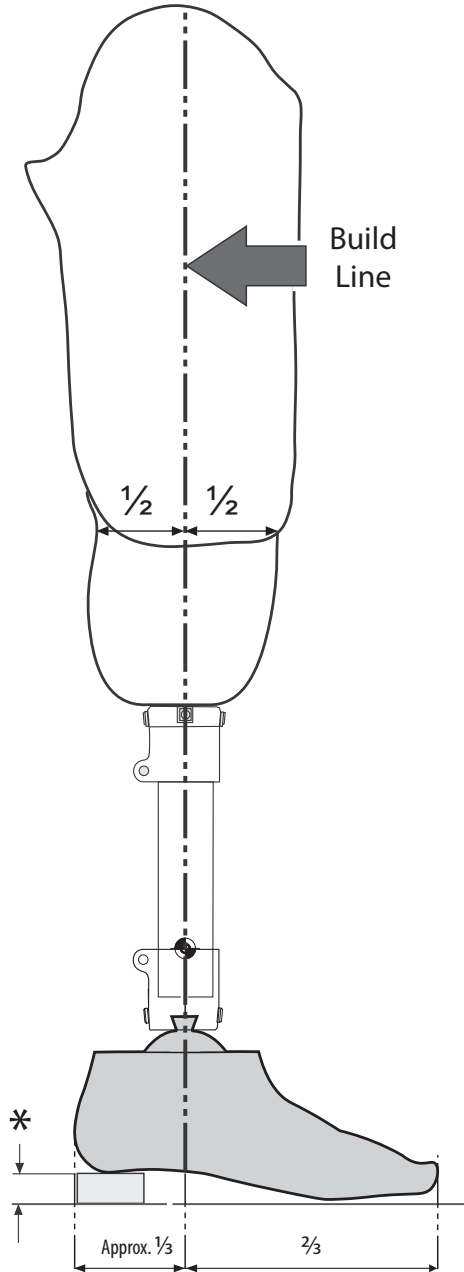

Align trans-femoral devices according to fitting instructions supplied with the knee, keeping the build line relative to the SuperSACH foot, as shown.

Static Alignment

With the flexion accommodated the build line should pass through the center of the socket in the sagittal (A-P) plane and through the SuperSACH foot, as shown. Make sure you accommodate the heel height of the shoe.

Dynamic Alignment

Coronal Plane

Make sure that M-L thrust is minimal by adjusting relative positions of the socket and foot.

Sagittal Plane

Check for a smooth transition from heel strike to toe-off. Make sure that when standing the heel and toe are evenly loaded and that both are touching the floor.

* Allow for users own footwear.

7 Fitting Advice

Symptoms	Reason	Action
<ul style="list-style-type: none"> Sinking at heel strike 	Heel too soft	Harden the heel cushion
<ul style="list-style-type: none"> Rapid transition from heel strike through stance phase Difficulty controlling heel action (e.g. foot jars into mid-stance) Foot feels too rigid 	Heel too hard	Soften the heel cushion

Using the Adjuster Screw


The adjuster screw is used to adjust the hardness of the heel cushion.

Note: Users who weigh between 100 kg and 125 kg may need maximum heel hardness. Users who weigh 60 kg or less may need minimum heel hardness.

To harden the heel cushion:

- Turn the adjuster screw **clockwise**.


Note: The heel cushion is set to maximum hardness when the screw is fully engaged.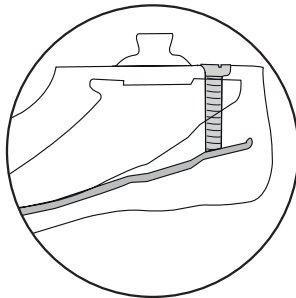

To soften the heel cushion:

- Turn the adjuster screw **counterclockwise**.

Note: The heel cushion is set to minimum hardness after 4 counterclockwise turns from the maximum hardness setting.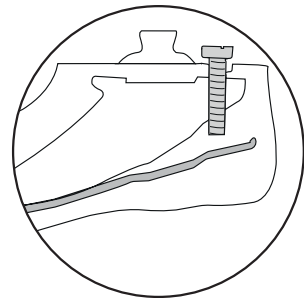

8 Bonding a Cosmesis

Before you begin

Lightly abrade and clean the mating surfaces with a suitable solvent cleaner.


938097

For instructions on fitting a cosmesis.

Use a contact adhesive — Evo Stik 528, Thixofix or an equivalent.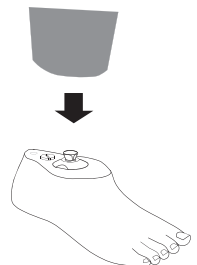


9 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight [size 26]:	449 g (15.1 oz)
Activity Level:	1–2
Maximum User Weight:	125 kg (275 lb)
Range of Adjustment:	±7 ° angular
Build Height:	See dimension 'A' below
Heel Height:	19 mm
Attachment:	Blatchford Female Pyramid Adapter

Fitting Length

Size (cm)	A
22	70 mm
23	
24	
25	75 mm
26	
27	80 mm
28	

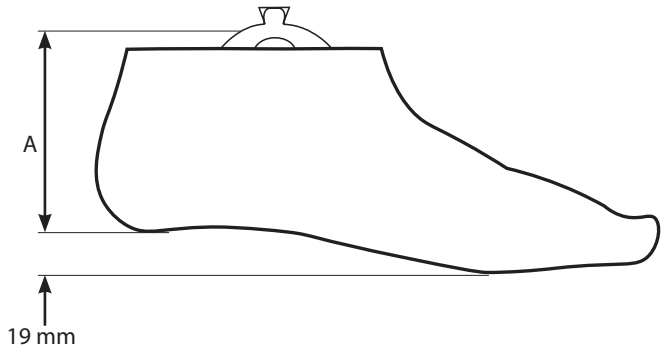10 Ordering Information

SuperSACH

Size (cm)	Part number — left	Part number — right
22	P529101	P529102
23	P529103	P529104
24	P529105	P529106
25	P529107	P529108
26	P529109	P529110
27	P529111	P529112
28	P529113	P529114

Ankle cosmesis

Size (cm)	Part number — left	Part number — right
All sizes	P561081	P561081

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

SuperSACH is warranted for 24 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd