Compact SAKL

Instructions for Use

P239143

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1 Description and purpose

The Compact Semi Automatic Knee Lock is a lightweight knee, ideally suited to individuals with low activity level who need the security of a locked knee during walking.

- Slim, low profile knee
- 140° knee flexion
- Locks automatically on extension
- Thigh release

Application:

- These instructions are for use by the Clinician/Practitioner
- The Compact SAKL is to be used exclusively as part of a lower limb prosthesis
- Intended for single user

Ensure that the user has understood all instructions for use, drawing particular attention to Section 3 Maintenance and Section 6 Unlocking procedure.

Activity level:

This device is recommended for users that have the potential to achieve Activity Level 1 or 2

Contra-indications.

Not for use with a turntable above the Compact SAKL.

Safety Information

The caution symbol highlights safety information which must be followed carefully

Any changes in performance e.g. unusual noise, stiffness of thigh release or reduced stability in stance should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

The Compact SAKL is not intended to be used in water or as a shower prosthesis. If the knee comes into contact with water wipe dry immediately.

Assembly and repair of the Product must only be carried out by suitably qualified practitioners or clinicians.

The user must not adjust or tamper with the setup of the Product.


The user should be advised to contact their clinician if their condition changes.


2 Fitting Instructions

Users to be aware of finger trap hazard at all times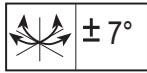


Blatchford:

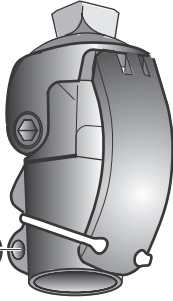
5
10Nm

Blatchford: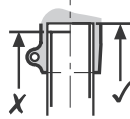

Rear view

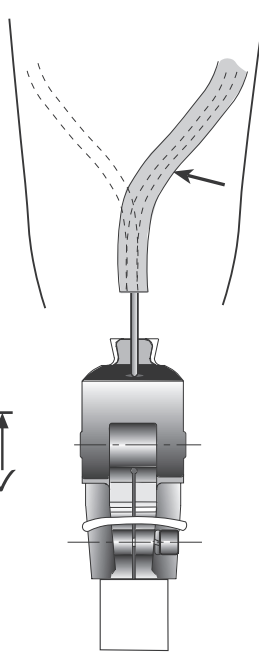938120

Tape entire length up to thigh release


Ensure the cable is not pinched in full flexion

After fixing cable run, ensure the lock engages fully and freely

Cosmesis
560032

SACH Ankle
Fairing
P561081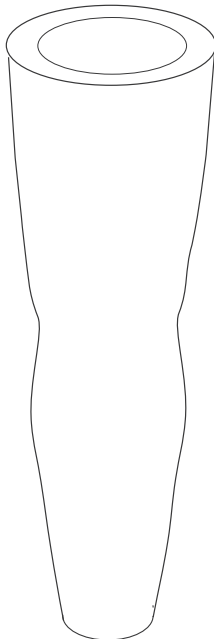

3 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check all non-adjuster screws for tightness, clean and reassemble as necessary
- If ML rock is evident, the blue plastic bearings 923700 may be renewed
see Maintenance section 1 Bearing Replacement
- If AP rock is evident, the buffers 234310 may be renewed
see Maintenance section 3 Buffer Replacement
- If the cable is found to have stretched, remove the operating lever knob and loosen the cable clamp set screw.

With the lever up, take up slack in cable without operating the pawl and retighten the set screw after applying Loctite 222. Cut excess cable if necessary, leaving enough for future adjustment, fold back into lever and refit knob.

See also 938120 Thigh Release Instructions for Use

The wearer should be advised:

Any changes in performance of this device must be reported to the practitioner


Changes in performance may include:

- Reduced support during stance
- Any unusual noise
- Stiffness or failure to release from lock in thigh release


Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers. Thoroughly dry before use.


1 Bearing Replacement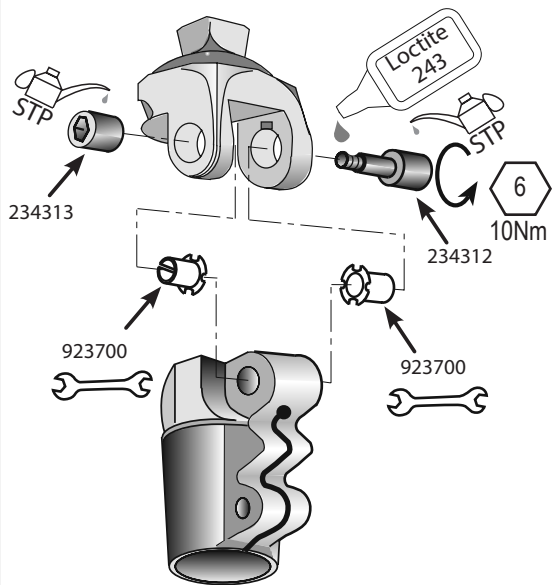


2 Patella Replacement

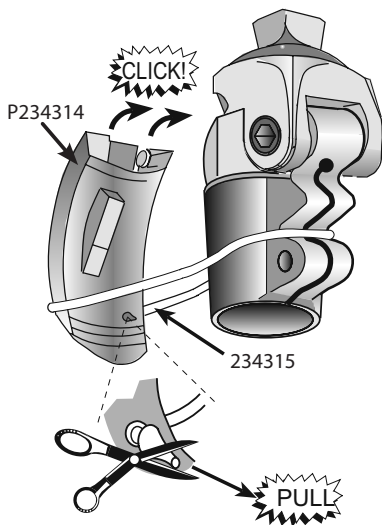3 Buffer Replacement

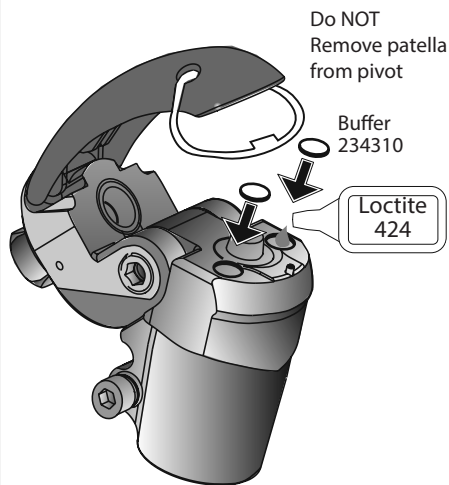4 Cable Replacement

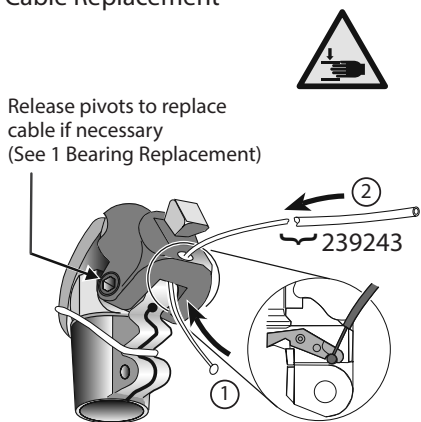4 Limitations on use

Intended life:

Service life of the product is covered by the warranty period - a local risk assessment should be carried out based upon activity and usage.

Lifting loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:


Avoid exposing the Compact SAKL to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use between -15°C and 50°C.

It is recommended that only Blatchford products be used in conjunction with the Compact SAKL.

5 Alignment




For socket manufacture use ESK height position on Dummy Knee Block on alignment mandrel.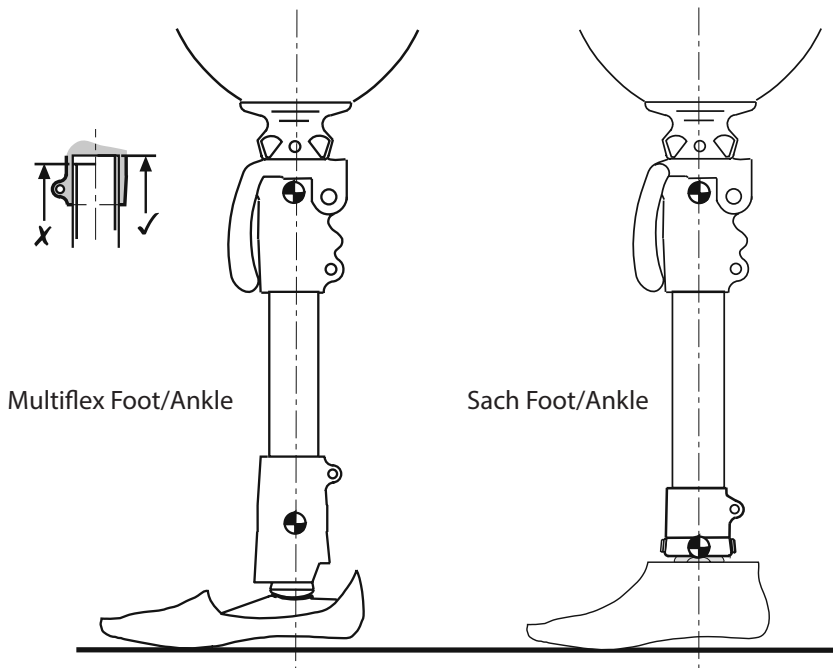

6 Fitting and set-up advice

Unlocking Procedure (Advice to User)


Instruct user that when releasing the lock ensure a flexion load is not being applied and the foot is on the ground.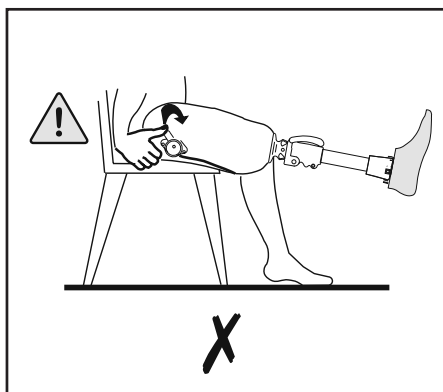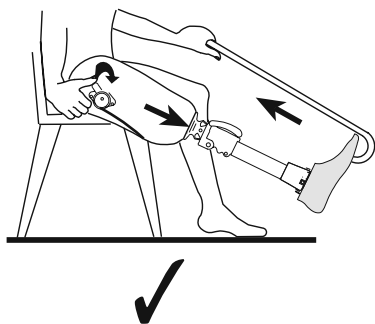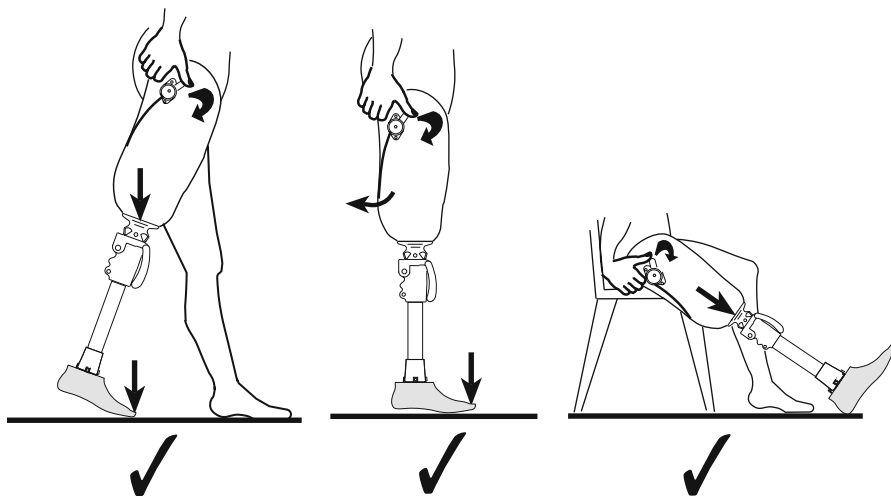

7 Technical Data

Product Ordering Code	SAKL
Material:	Aluminium, ST STL
Operating and Storage Temperature Range:	-15°C to 50°C 5°F to 122°F
Component Weight:	325g (12ozs)
Recommended Activity Level:	1/2
Maximum User Weight:	125kg (275lb)
Proximal Alignment attachment:	Male Pyramid (Blatchford)
Distal attachment	30mm Diameter
Build Height:	40mm
Proximal pyramid to distal tube end [See diagram below]	

Fitting Length (mm)

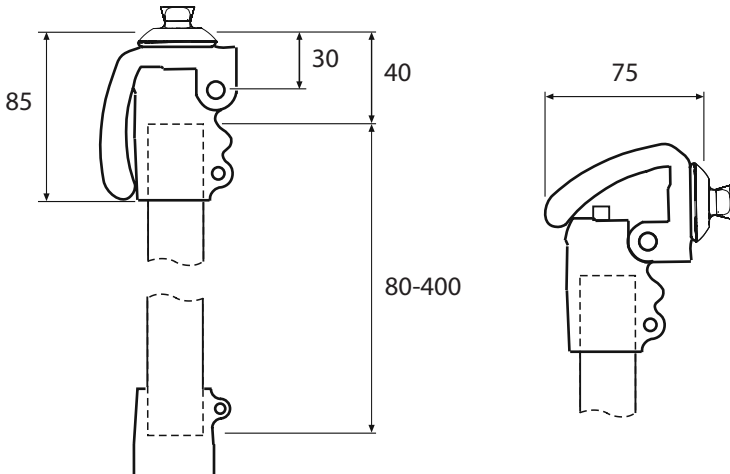Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The warranty for the Compact SAKL is 24 months from original date of purchase unless otherwise stated.

The user should be aware that changes or modifications not expressly approved will void the warranty. See catalogue for details.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

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Blatchford SAS
Endolite India Ltd